Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primalabsa.ch
primalabsa.ch

EC REP QbD Rep5 BV
Groenenborgerlaan 16
2610 Wilrijk - Belgium

800017_IFU_11_8.0

11/2024

REF 800017-x

Registered trademarks.

INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
NOTICE D'UTILISATION
ISTRUZIONI PER L'USO

HELICOBACTER PYLORI TEST

Self-test for the determination of anti *Helicobacter pylori* antibodies in whole blood samples

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Huailan Tianda Medical Instruments Co., Ltd., No.106 East Songjiang Road, Huaiyin, Economic&Technological Development Zone, 223300 - Huailan City, Jiangsu - P.R. China

Riomavix Sociedad Limitada
Calle de Almansa 55, 1D
Madrid 28039, Spain

Alcohol cleansing gauze
Alkohol-Reinigungsgaze
Gaze nettoyante à l'alcool
Garza detergente alcolica

Phoenix Innovative Healthcare Manufacturers Pvt. Ltd.,
El-209, Shi Mahape Road, Electronic Zone, MIDC,
TTC Industrial Area, Mahape, Navi Mumbai
400 710 MH I India

Advena Ltd., Tower Business Centre,
2nd Flr, Tower Street, Swatara,
BKR 4013, Malta

SYMBOLS / SYMBOLE / SYMBOLES / SIMBOLOGIA

IVD In vitro diagnostic device In-Vitro-Diagnostikum Dispositif médical de diagnostic in vitro Dispositivo medico-diagnostico in vitro	TEMP Temperature limits Temperaturbegrenzung Limites de température Limiti di temperatura	REF List number Liste-nummer Code produit Codice prodotto
READ Read the instructions before use Beschriften Sie die Gebrauchsanweisung Consultez le mode d'emploi Leggere attentamente le istruzioni per l'uso	DO NOT REUSE Do not reuse Nicht wiederverwenden Ne pas réutiliser Non-réutiliser	LOT Lot number Chargenbezeichnung Code du lot Lotto Prodotto
STERILE R Sterilized using irradiation Sterilisation durch Bestrahlung Méthode de stérilisation utilisant l'irradiation Sterilizzato mediante radiazione	SUFFICIENT Sufficient for <n> tests Ausreichend für <n> Tests Suffisant pour <n> tests Sufficiente per <n> test	CE CE marking CE-Kennzeichnung Marquage CE Marchio CE
EC REP Authorised Representative in the European Community Bevollmächtigter in der Europäischen Gemeinschaft Mandatado dans la Communauté européenne Mandatario nella Comunità Europea	EXPIRY DATE Expiry date (last day of the month) Verwendbar bis (letzter Tag des Monats) Utiliser jusqu'en (dernier jour du mois) Utilizzare entro (ultimo giorno del mese)	LEGAL MANUFACTURER Hersteller Fabricant Fabbricante

ENGLISH

HELICOBACTER PYLORI INFECTION AND SPECIFIC IgG

Helicobacter pylori is a bacterium whose ideal habitat is the human gastric mucosa. The infection is often asymptomatic, but it can sometimes cause gastritis and ulcers in the stomach or in the duodenum segment of the small intestine. The most common symptoms of these conditions are heartburn or pain in the upper part of the abdomen, especially if fasting or in the early morning, when the stomach is empty. Other associated symptoms include nausea, feeling full, reflux, loss of appetite, burping, weight loss and diarrhoea. Ulcers can sometimes bleed, causing anemia over long periods of time. In the long term, H. pylori infection is associated with an increase in the risk of gastric cancers. The simplest way to detect infection is by checking for the presence of specific antibodies (IgG) that are produced by the immune system after coming into contact with the bacterium and trying to eliminate it. The *Helicobacter pylori* Test can detect the presence of these antibodies in a few minutes.

PRINCIPLE OF THE TEST

Helicobacter pylori Test is an immunochromatographic test that detects the presence of anti-*Helicobacter pylori* IgG antibodies in the blood, using special monoclonal antibodies conjugated with gold and integrated into the reactive strip.

REFERENCES

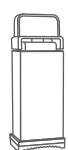
- Mati Moyat, Dominique Velin. Immune responses to *Helicobacter pylori* infection. World J. Gastroenterol. 2014 may 21; 20(19): 5583-5593IgE." *Journal of Allergy and Clinical Immunology*. 2014;133(2):589-91.
- Khean-Lee Goh, Wah-Kheong Chan, Seiji Shiota and Yoshio Yamaoka. Epidemiology of *Helicobacter pylori* Infection and Public Health Implications. *Helicobacter*, 2011 Sep;16(0 1):1-9.
- Kyichi Adachi, Tomoko Mishiho, Shino Tanaka and Yoshikazu Kinoshita. Analysis of negative result in serum anti-pylori IgG antibody test in cases with gastric mucosal atrophy. *J. Clin Biochem Nutr*.2016 Sep;59(2):145-148.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.


- * hermetically sealed aluminium pouch containing:
1 HELICOBACTER PYLORI TEST cassette
1 desiccant bag

- * sterile lancets for self-drawing blood

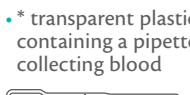- * vial with dropper tip containing sufficient HELICOBACTER PYLORI TEST diluent for 1 test

- * transparent plastic bag containing a pipette for collecting blood

- * alcohol cleansing gauze
- * 1 instructions for use leaflet

- Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
- Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
- The desiccant packet must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

- Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
- Keep the Test out of the reach of children.
- Do not use the Test after the expiry date or if the package has been damaged.
- Follow the procedure exactly, using only the specified quantities of blood and diluent.
- Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
- Use the test and lancing device once only.
- The test is for external use only. DO NOT SWALLOW.
- In vitro diagnostic device for individual use.
- Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
- After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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