
Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
support@primlabsa.ch
primlabsa.ch

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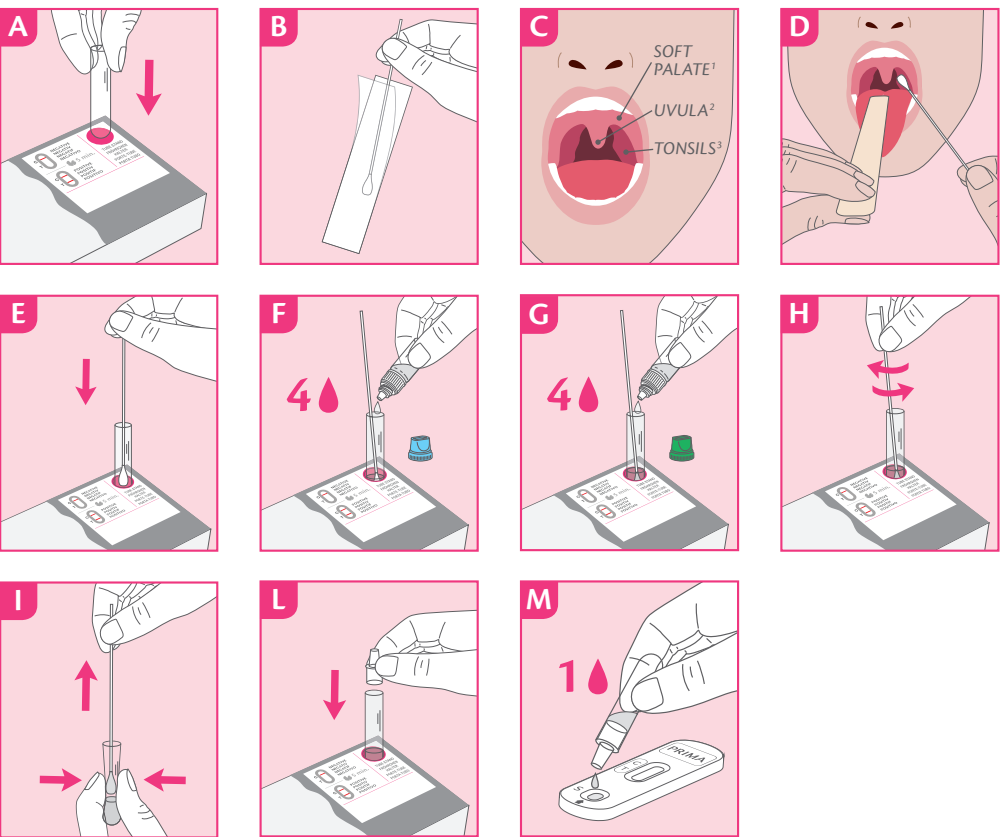1. Banerjee S, Ford C, "Rapid Tests for the Diagnosis of Group A Streptococcal Infection: A Review of Diagnostic Test Accuracy, Clinical Utility, Safety, and Cost-Effectiveness", Ottawa (ON): Canadian Agency for Drugs and Technologies in Health; 2018 May. CADTH Rapid Response Reports.
2. Emily H. Stewart, Brian Davis, B. Lee Clemans-Taylor, Benjamin Litzenberg, Carlos A. Estrada, and Robert M. Centor, Rapid Antigen Group A Streptococcal Test to Diagnose Pharyngitis: A Systematic Review and Meta-Analysis, *PLoS One*. 2014; 9(11).
3. Llor C, Bjerrum L, Munk A, Cots JM, Hernández S, Moragas A, HAPPY ADAPT Investigators. "Access to point-of-care tests reduces the prescription of antibiotics among antibiotic-requesting subjects with respiratory tract infections." *Respir Care*. 2014 Dec.

SYMBOLS / SYMBOLE / SIMBOLOGIA / SIMBOLOS / SYMBOLE / SYMBOLES / SYMBOLO / SYMBOLO

IVD In vitro diagnostic device / In-Vitro-Diagnostikum / Dispositif médical de diagnostic in vitro / Dispositivo medico-diagnostico in vitro / Prodotto sanitario diagnostico "in vitro" / Wyposażenie do diagnostyki in vitro / In-Vitro-diagnostisch hulpmiddel / Enhet för in-vitro-diagnostik / Diagnostische zariadení in vitro

STERILE R Read the instructions before use / Beachten Sie die Gebrauchsanweisung / Consulter le mode d'emploi / Leggere attentamente le istruzioni per l'uso / Lea atentamente las instrucciones antes de su uso / Przel czytaj uważnie instrukcję / Lees de instructies voor gebruik / Läs instruktionerna före användning / Před použitím si přečtěte návod

STERILE EO Sterilized using irradiation / Sterilisation durch Bestrahlung / Méthode de stérilisation utilisant l'irradiation / Sterilizzato mediante radiazione / Esterilizado utilizando irradiación / Wysterylizowano przez napromienianie / Gesteriliseerd door middel van bestraling / Steriliserad med strålning / Sterilizováno ozářením

STERILE EO Sterilized using ethylene oxide / Sterilisiert mit Ethylenoxid / Sterilisé avec de l'oxyde d'éthylène / Sterilizzato mediante Ossido di Etilene / Esterilizado con óxido de etileno / Wysterylizowano przy użyciu tlenku etylenu / Gesteriliseerd met ethylenoxide / Steriliserad med etylenoxid / Sterilizováno etylenoxidem

Expiry date (last day of the month) / Verwendbar bis (letzter Tag des Monats) / Utiliser jusqu'au (dernier jour du mois) / Utilizzare entro (ultimo giorno del mese) / Utilizar hasta (último día del mes) / Termin ważności (ostatni dzień miesiąca) / Vervaldatum (laatste dag van de maand) / Sällastigingsdatum (sista dagen i månaden) / Datum expirace (poslední den měsíce)

CE Legal manufacturer / Hersteller / Fabricant / Fabricante / Fabricante / Producent / Juridische fabrikant / Juridisk tilvarekør / Legální výrobce

CE CE marking / CE-Kennzeichnung / Marquage CE / Marchio CE / Marcado CE / Znak CE / CE-markering / CE-märkning / Značka CE

Do not reuse / Nicht wiederverwenden / Ne pas réutiliser / Non riutilizzare / No reutilizar / Nie używać ponownie / Niet opnieuw / Återanvänd inte / Nepouzívejte opakovaně

S Sufficient for <= tests / Ausreichend für <= Tests / Sufficient pour <= tests / Sufficiente per <= tests / Sufficiente per <= tests / Sufficiente per <= tests / Wystarczająca na następującą liczbę testów -> / Voldoende voor <= tests / Tillräckligt för <= tester / Dostatečné pro <= testů

REF List number / Katalognummer / Code produit / Codice prodotto / Código producto / Numer wykazu / Listnummer / Listnummer / Číslo seznamu

LOT Lot number / Chargenbezeichnung / Code du lot / Lotto prodotto / Lote de producto / Numer partii / Lotnummer / Partinummer / Číslo šarže

EC REP Authorised Representative in the European Community / Bevollmächtigter in der Europäischen Gemeinschaft / Mandatario nella Comunità europea / Mandatário na Comunidade Europeia / Reprezentante autorizado en la Comunidad Europea / Autorizovaný zástupce v teritoriu Wspólnoty Europejskiej / Gemachtigter vertegenwoordiger in de Europese Gemeenschap / Autoriserad representant i Europeiska gemenskapen / Autorizovaný zástupce v Evropském společenství

MEDICAL DEVICES INSIDE THE KIT / MEDICINSKE GERÄTE INNERHALB DES KITS / DISPOSITIFS MÉDICAUX À L'INTÉRIEUR DU KIT / DISPOSITIVI MEDICI INTERNI AL KIT / PRODUCTOS SANITARIOS DENTRO DEL KIT / WYROBY MEDYCZNE W ZESTAWIE / MEDICINSKE HULPMIDDELEN IN DE KIT / MEDICINSKA ENHETER I KITTET / ZDRAVOTNICKÁ ZAŘÍZENÍ V SADĚ

STERILE R Sterile swab / Steriliser Tupper / Tampon stérile / Tampone sterile / Hisopo estéril / Sterylina wymazówka / Steriel wattenstaafje / Steriel bomulspringe / Sterilni štětkica

STERILE EO Sterile swab / Steriliser Tupper / Tampon stérile / Tampone sterile / Hisopo estéril / Sterylina wymazówka / Steriel wattenstaafje / Steriel bomulspringe / Sterilni štětkica

STERILE EO Sterile tongue depressor / Steriliser Zungenspatel / Abaisse-langue stérile / Abassalingua sterile / Depressor de lingua estéril / Steryliny depresor języka / Steriel tongepressor / Sterilni lingpressor / Sterilni lopatica na jazyk

STERILE EO Applimat SA
21, Route de la Plan 1,
CH-1618 Ostail-Saint-Denis - Switzerland

STERILE EO Darsu AS - Sandbakken
1-3, Gøttestoe, DK-3660 Steenløse - Denmark

STERILE EO Huachenyang (shenzhen) Technology Co., Ltd.
128th Shengren East Avenue Guangyuan Community, Xinqiao St, Bao'an, Shenzhen, Guangdong, China

STERILE EO R Right B.V.,
Roald Daalman 47, 5629 MC Eindhoven, Netherlands

STERILE EO Jiangsu Suyun Medical Materials Co., Ltd.
222002 Liangyunqiang, Jiangsu Province - People's Republic of China

STERILE EO Shanghai International Trading Corp. GmbH (Europe)
Eiffelstrasse, 80, D-20537 Hamburg - Germany

PRIMA HOME TEST

STREP A TEST

Self-test for the detection of group A β -hemolytic Streptococcus in throat swab
Selbsttest zum Nachweis β -hämolytischer Streptokokken der Gruppe A im Rachenabstrich
Auto-test pour détecter la bactérie Streptococcus β -hémolytique du groupe A dans un prélèvement de gorge
Self-test per la rilevazione del batterio Streptococco β -emolitico di gruppo A da tampone faringeo
Autoprobe para la detección de estreptococos β -hemolíticos del grupo A en frotis faringeo
Autotest do wykrywania paciorkowców β -hemolizujących grupy A w wymazie z gardła
Zelftest voor de detectie van groep A β -hemolytische streptokokken in keelswab
Självtest för påvisande av grupp A β -hemolytiska streptokocker i halsprov
Vlastní test na detekci β -hemolytických streptokoků skupiny A ve výtěru z krku

EN - INSTRUCTIONS FOR USE

STREPTOCOCCUS A

Group A β -hemolytic Streptococcus (also known as *Streptococcus pyogenes*) is a Gram-positive bacterium that causes a variety of pathological conditions and complications, including pharyngitis (throat infection), skin infections (such as impetigo and erysipelas), acute glomerulonephritis, sepsis and rheumatic heart disease. Pharyngitis is commonly caused by viral infections and less often by bacterial infections, which require treatment with antibiotics. STREP A TEST is an immunological test designed to specifically identify Streptococcus A bacteria in pharyngeal infections. It helps to understand quickly if a sore throat is caused by Streptococcus A or by other microorganisms that do not require treatment with antibiotics.

PRINCIPLE OF THE TEST

STREP A TEST is an immunochromatographic test that detects the presence of Streptococcus A specific antigens, using gold-conjugated antibodies integrated into the reactive strip.

FAQ - QUESTIONS AND ANSWERS

HOW DOES THE STREP A TEST WORK? Group A β -hemolytic Streptococcus could be involved in infections of the pharynx. For appropriate antibiotic treatment to be administered, the bacterium must be identified first. STREP A TEST can detect Streptococcus A antigens through the specific antibodies it contains.

WHEN CAN THE TEST BE USED? STREP A TEST can be carried out if there are symptoms such as pain when swallowing, sore throat, red and swollen tonsils (sometimes with white spots or pus), small red patches on the back of the palate (soft or hard), swollen lymph nodes, fever, headache, nausea or vomiting (especially in children).

CAN THE RESULT BE INCORRECT? The result is correct if the instructions are followed carefully. However, the result may be incorrect if STREP A TEST is used before performing the test or if the sample from the pharynx is not collected and prepared correctly (carefully follow the instructions for collecting the sample from the pharynx - fig.D). Incorrect results may also be obtained if the test is carried out after treatment with antibiotics or at the beginning of the infection (the concentration of bacteria is below the level of detection).

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the Test band, even if it is weak.

WHAT IS THE PURPOSE OF THE CONTROL BAND? The appearance of this band means the test has worked correctly.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive, Streptococcus A has been detected in the sample from the pharynx and you must consult a physician. Please be aware the STREP A TEST cannot distinguish between live and dead bacteria. Patients who have recently recovered from Streptococcus A or similar infections may test positive because of inactive bacteria that are still present in the body.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the symptoms were not caused by the Streptococcus A or that the sample did not contain a concentration of bacteria that was high enough to be detected by the test. If symptoms persist, seek medical advice.

IS THE RESULT RELIABLE IF READ AFTER 10 MINUTES? No. The test must be read 5 minutes after the procedure is completed and no later than 10 minutes from this time.

HOW ACCURATE IS THE STREP A TEST? The device specifically identifies Group A β -hemolytic Streptococcus when its concentration in the extracted sample is equal to or higher than 10³ CFU/mL (cut-off value). In some cases, samples with concentrations below this value (down to 5 x 10² CFU/mL) can yield a positive result. The test is very accurate. Assessment reports show an analytical specificity of 99.00% (CI 95% 97.63 - 100%) and a sensitivity of 91.89% (CI 95% 96.08 - 97.20%) with an overall concordance rate of 97.90% (CI 95% 96.08 - 99.72%). Although this test is reliable, it may yield false positive and negative results.

PRECAUTIONS

1. Read these instructions for use carefully before performing the test. The test is reliable if the instructions are followed correctly (reaction times, how the sample is collected and the extraction instructions shown).
2. Keep the Test out of the reach of children.
3. Not suitable for use on children under the age of 2 years.
4. Do not use the Test after the expiry date of the package has been damaged.
5. Store the Test components at a temperature between +4 °C and +30 °C. Do not freeze.
6. Use individual Tests only once.
7. In vitro diagnostic device for individual use.
8. The test is for external use only. Do not swallow solutions 1 and 2 and the other items in the box. If swallowed, seek medical advice immediately. Show the remaining parts of the box, the instructions for use and the package.
9. If liquids come into contact with the eyes, skin or other mucousae, rinse immediately and thoroughly with water and contact a physician. Show the vials' labels.
10. After using, dispose of all components according to local laws. Liquid wastes should not be disposed of down the drain.
11. Do not open the sealed elements (test case, swab and tongue depressor) until just before performing the test. Take care to open as marked.
12. Do not switch the caps of the reagent vials.
13. All kit components are single-use, with the exception of the test package (which can be re-used as test tube holder) instructions for use and solution 1 and 2, which must be stored, closed with their caps, at a temperature between +4°C and +30°C.

CONTENT

- * The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.
- * hermetically sealed aluminium pouch containing 1 STREP A TEST cassette, 1 desiccant bag * sterile swab * sterile wooden tongue depressor * empty plastic test tube for sampling with dropper 1 instructions for use leaflet 1 dropper vial (Blue cap) containing enough STREP A TEST - R1 solution to perform * test. R1 solution contains 20.7% Sodium Nitrite; 1 dropper vial (Green cap) containing enough STREP A TEST - R2 solution to perform * test. R2 solution contains 0.55% Acetic Acid.

Danger H302: Harmful if swallowed.
Precaution P264: Wash hand thoroughly after handling.
P270: Do not eat, drink or smoke while using this product.

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